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Tourniquet „PAX Extremities Tourniquet“

The „PAX-Extremities-Tourniquet“ (P-E-T) is a winch binding system for heavy bleeding at the extremities. Due to its variable length and the winch system with locking points at every 90° per revolution, it is equally applicable for arms and legs. Due to manual pre-tensioning when applying the tape, only a few turns are required to ensure the setting through the PAX Extreme Tourniquet. On the writing surface at the end of the tape, the application time and other information can be noted.

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Date: 17.08.2018

Warning

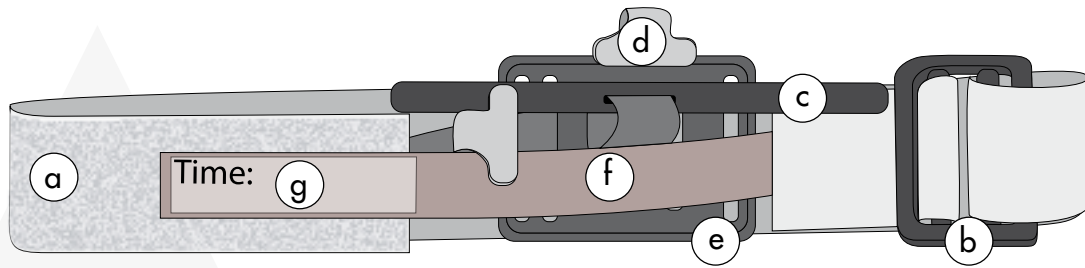
The P-E-T may only be used by professional users.

Limitation of liability

It is mandatory to read and understand the instruction for use carefully and completely before first use. The PAX-Extremities-Tourniquet (P-E-T) should only be used as directed by military service operational guidelines, the EMS authority, or under the supervision of a medical practitioner. According to the situation, the application of the P-E-T should be defined as the best alternative treatment.

Recommendation

The P-E-T unavoidably loses durability over the course of its shelf life. This is accelerated by UV rays, heat, chemicals, harsh atmospheres and dirt. Please check before you use the P-E-T to ensure that it is in proper working condition. The P-E-T must be in a proper technical and hygienic condition, otherwise a secure application can not be guaranteed.

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Component Description

- a belt with velcro
- b closure buckle
- c P-E-T-rod
- d fixation points of the P-E-T-rod-plate
- e P-E-T-rod-plate
- f red security belt
- g labelling field for time of application

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1. Purpose

The PAX-Extremities-Tourniquet (P-E-T) is designed to stop heavy, life-threatening bleeding of the upper and lower extremities of adults and teens if,

- bleeding is unmanageable in other ways
- according to the situation, the application of the P-E-T is the best alternative treatment.

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2. Introduction

It is mandatory to read and understand the instructions for use carefully and completely before first use.

The PAX-Extremities-Tourniquet should only be used as directed by military service component guidelines, the EMS authority, or under the supervision of a medical practitioner.

According to the situation, the application of the P-E-T should be defined as the best alternative treatment.

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3. Unintended Side-Effects and Misapplication

Applying the P-E-T can cause severe pain to the patient. In rare cases inappropriate use of the P-E-T can caused serious injuries or loss of limbs.

The P-E-T is a single-use product. Use of the P-E-T several times is strictly prohibited.

Inadequate performance and material fatigue could be a result of using the P-E-T several times.

Furthermore the risk of a contaminated P-E-T is much higher for both user and patient.

Using the P-E-T several times could lead to the P-E-T rod loosening itself during transport or patient care.

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4. General Information for Use

1. Control the function of the P-E-T during transport and react appropriately if the wound starts bleeding again.
2. If bleeding recommences, twist the P-E-T-rod again until bleeding has stopped. Lock the P-E-T-rod with the red safety belt as described below.
3. To enhance the function of the P-E-T, you must elevate the affected extremities.
4. Use of the P-E-T should not cause interference during X-Ray or CT scan procedures. The material of the P-E-T is radiolucent.

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5. Storage

- store at room temperature
- store dry and protected from light
- avoid contact with aggressive substances / chemicals

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6. Inspection and Durability

The P-E-T unavoidably loses durability over the course of its shelf life. This is accelerated by UV rays, heat, chemicals, aggressive environmental conditions and dirt. Ensure yourself of a proper condition of the P-E-T before use. The P-E-T must be in a perfect technical and hygienic condition, otherwise a safe application is not guaranteed.

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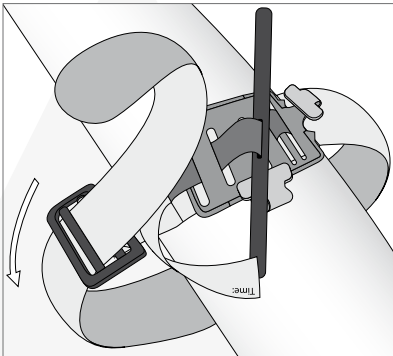
7. Disposal


You can deposit the P-E-T in the non-recyclable waste after proper disinfection. Please note the local requirements for disposal of non-recyclable waste.

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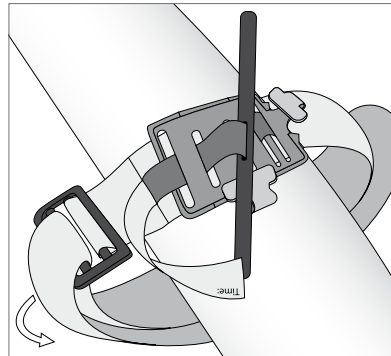
Insert the wounded arm through the loop of the P-E-T

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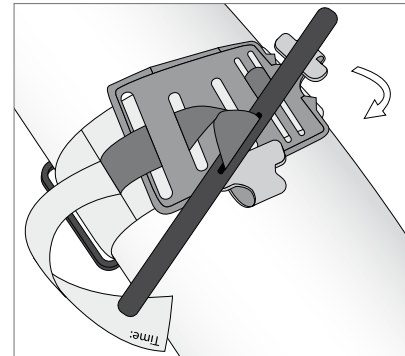
Pull the belt tight and securely and fasten the belt back on itself with velcro

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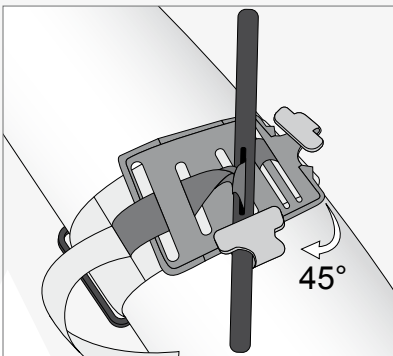
Twist the Tourniquet rod until the bleeding has stopped

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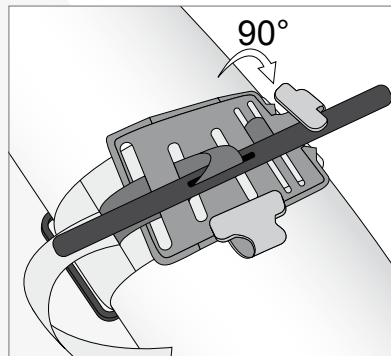
Lock the Tourniquet rod with one of the two fixation hooks

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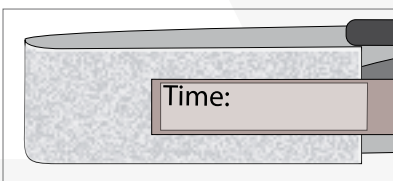Secure the tourniquet rod with the red security belt

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
Record the time of application on the red security belt

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
The PAX-Extremities-Tourniquet should be applied above the wound, between elbow & shoulder.

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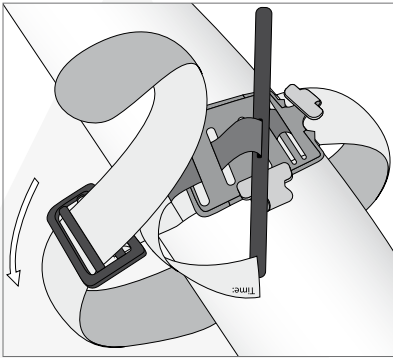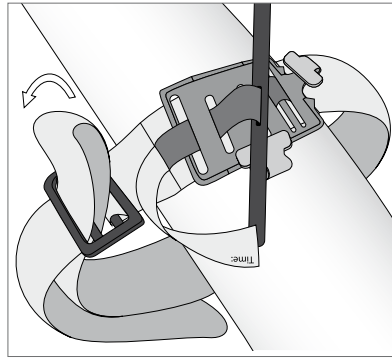
Lead the belt around the leg and pass the end of the belt through the inside slit of the closure buckle

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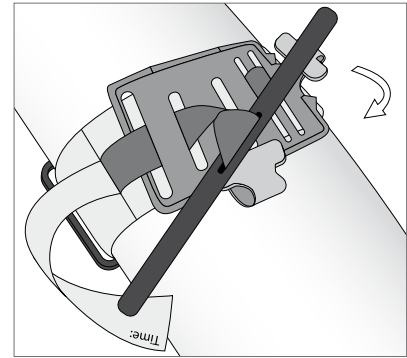Pass the end of the belt through the outside slit of the closure buckle and pull the belt tight. Securely fasten the belt back on itself with velcro

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X-CEN-TEK GmbH & Co. KG