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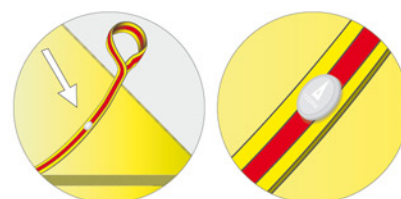
The tip of the triangle of the PAX-logo print onto the PAX-Rescue-Boa marks the centre of the PAX-Rescue-Boa.

Hinweis: Position PAX Seriennummer
Hint: Position of PAX Serial number

Quick evacuation of the patient by means of the **PAX-Rescue-Boa** tested according to **DIN/EN1865**. Delivery is carried out in a practical protection bag. In case of emergency the incorporated fleece blanket can be used for conservation of body heat.

Item indications: PAX-Rescue-Boa

Item description	PAX-Rescue-Boa
Item number	118171908
Colour	high-viz yellow
Material	Outside: PES Polyester FR with dirt-repellant polymer coating Inside: Polyester with polygene
Length	315 cm
Weight	0,9 kg
Loading capacity	ca. 200 kg (horizontal)
Test standard	DIN/EN 1865
Classification	Medical product class I
Temperature range	Storage between +10 degree - +30 degree Celsius Application between -10 degree - +50 degree Celsius
Serial number	positioned on the belt strap of the hand loop

X-CEN-TEK GmbH & Co. KG
Westerburger Weg 30
26203 Wardenburg / Germany
Phone: +49 (0)4407 - 7 14 76 0
Fax: +49 (0)4407 - 7 14 76 99
E-mail: info@pax-bags.de
Web: www.pax-bags.de

Manufacturer / Sales and
Distribution

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Material: PAX-Rescue-Boa

The PAX-Rescue-Boa consists of a high-viz yellow coating out of 100% polyester with a waterproof polyurethan coating, a rolled up antibacteria fleece blanket and a special coated belt strap for the hand loops.

Coating	<i>The coating out of polyester is bacteria and virus proof, flame retardant according to DIN EN ISO 12952 -1 & -2, suitable for chemical washing, dryer proof, steam sterilisable and suitable for dry cleaning and resistant against blood, urine, oils and fats as well as against surface disinfectants.</i>
Fleece blanket	<i>The fleece blanket consists of 100% polyester with antibacteria properties.</i>
Lifting sling with carrying sling	<i>The hand loops are manufactured from a coated belt strap, are easy to clean and also suitable for wipe disinfection with all common surface disinfectants.</i>

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Intended purpose

The PAX-Rescue-Boa is used in order to evacuate a patient/person from an environment that makes a purposeful work impossible or when time pressure demands it. Moreover it serves to optimally immobilise the cervical and thoracic spine during and by proper use under the present circumstances. The application is restricted to the pre-hospital care and to the first phase of clinical care. Open wounds, fractures and injuries have to be treated according to the current rules of the specialty societies prior to the application.

This product is about a medical product which is only to be used by briefed and regularly product trained skilled-personnel. The type of application follows the current, generally recognized medical directives and recommendations of the corresponding specialty societies.

Furthermore the incorporated fleece blanket can be used as emergency warmth blanket.

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Reusability:

The product can be reused when functionality is maintained and when the material is undamaged. Prior to a reapplication the user has to check and ensure the proper condition of the product. Only products of proper technical and hygienic condition may be reused.

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The product has a lifetime of max. 5 years from purchase date. Every year of use the person in charge of medical products has to carry out a visual inspection of all seams, connections, handles and surface material and to record it at the end of these instructions for use. Defective products may not be used anymore.

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Intended use

Basically the PAX-Rescue-Boa may only be used in combination with a cervical collar correctly adjusted to its size (see series of pictures). When technically correctly used the PAX-Rescue-Boa allows an optimal immobilization of the patient's spine under the present circumstances.

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Indication:

The PAX-Rescue-Boa can be used in order to evacuate a patient with a body height of 150 cm and more from his environment. The rescue sling serves as immobilization of the cervical and thoracic spine during the evacuation of the patient. Another field of application is the conservation of body heat of the patient. Once taken the PAX-Rescue-Boa out of its cover it can be unrolled after opening the velcro fastener. After this procedure a square blanket surface with an edge length of 2,15 m appears.

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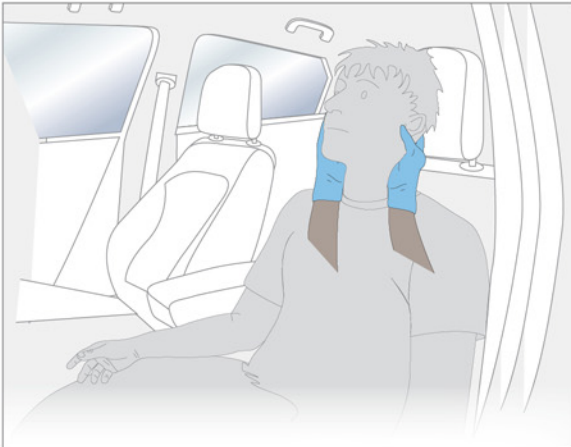
Contraindication:


The PAX-Rescue-Boa may not be used when by means of its use the patient could be harmed more than acceptable under the circumstances. This e.g. counts for foreign objects located in those areas of the body that would get in direct contact with the PAX-Rescue-Boa.

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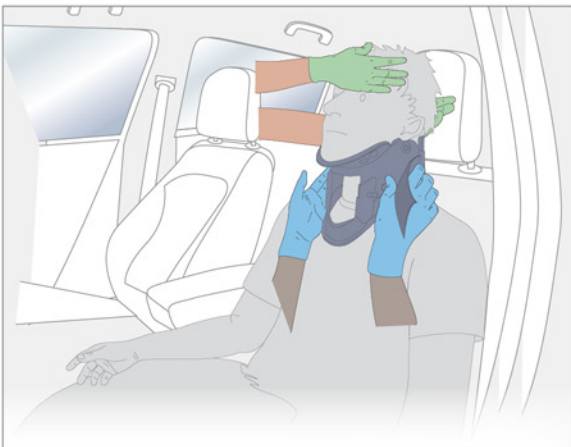In order to immobilize the cervical and thoracic spine right from the beginning, it is indicated to have the patient's head secured by helper 1. Then helper 2 can take over this immobilization from the corresponding alternative vehicle side as well as from the back seats.

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Thus then helper 1 can fix a cervical collar to the patient. In doing so attention is to be paid to the right size and to the correct fixing.

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Now the centre mark (tip of the red triangle in the PAX logo) is positioned in front of the larynx of the patient by helper 1 and both ends of the PAX-Rescue-Boa are lead over the shoulders to the back.

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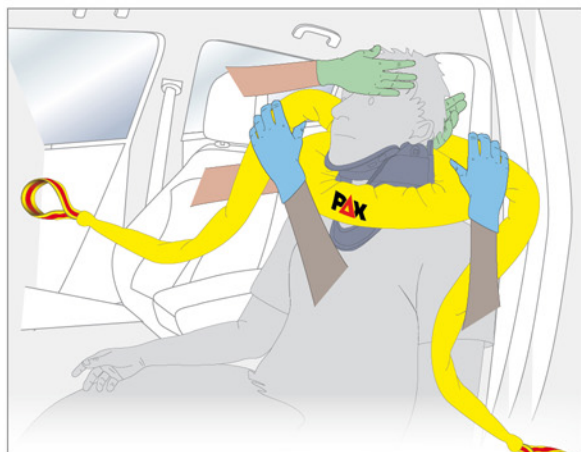
Also important at this stage is the immobilization of the head to be ensured by helper 2.

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
Both ends of the PAX-Rescue-Boa are now crossed at the dorsal area of the cervical spine by helper 1. Now helper 1 pulls open the PAX-Rescue-Boa in order to ensure immobilization of the cervical spine.

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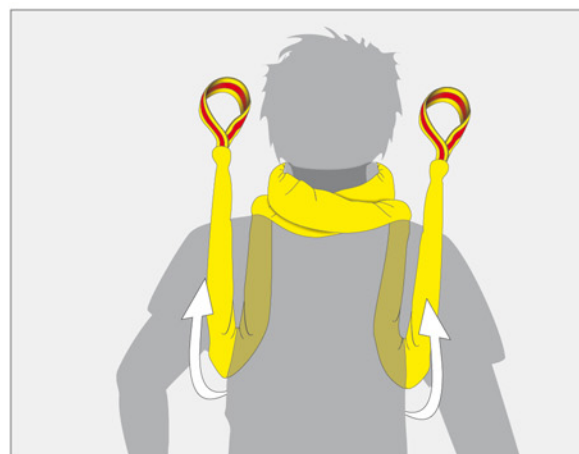Then the ends of the PAX-Rescue-Boa are brought to the ventral area and are lead under the particular armpit to the back.

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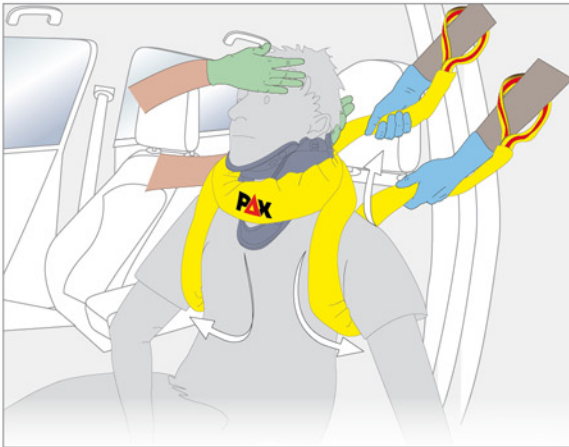
Now helper 1 prepares the spine board (or other comparable rescue systems). It must be secured from sliding away and has to be positioned slightly below the upper edge of the seat.

Especially with comatose patients it is recommended to use the PAX-immobilization-hand loop (see following pages). It hinders the arms drifting apart during further rescue measures.

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Helper 1 now reaches for the PAX-Rescue-Boa through both safety slings so that the hands remain above the shoulders. The tractive forces to generate now to the PAX-Rescue-Boa immobilize the cervical spine of the patient in such a way that manual immobilization of his head can be stopped by helper 2.

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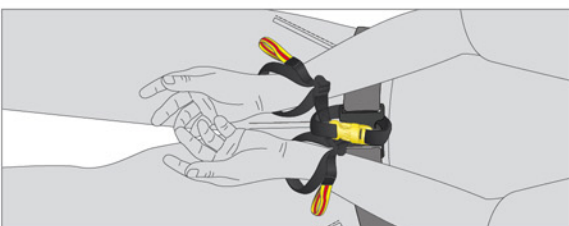
Now helper 1 pulls the patient onto the prepared rescue device; doing that helper 2 supports the rotation of the body.

In order to lay down the patient axis-aligned onto the rescue device helper 2 controls and corrects the body position of the patient during the pulling process.

Since a hyperflexion of the cervical spine could harm the patient attention is to be paid to the patient's head being able to freely slide towards the cranial without being lifted.

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When the patient is properly secured to the rescue device the PAX-Rescue-Boa can be loosened. The untieing process is especially easy and quickly to do thanks to the shape getting thinner and thinner coming to the ends.

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Fixation of the wrists from the patient by means of the PAX-Immobilization-hand loop guarantees a constant positioning of the arms.

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In addition to the PAX-Rescue-Boa (but also applicable with other rescue devices) the PAX Immobilization-hand loops are available.

Especially with comatose patients it is necessary to position the arms tight to the body. In order to avoid arms hanging down in an uncontrolled way these can be easily fixed with the PAX Immobilization hand loop.

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When locked the PAX Immobilization-hand loop consists of three single loops.

Determined to be fixed to belt loops of the trousers or directly to the belt. It is fitted with a PAX snap-lock buckle for easy opening and locking.

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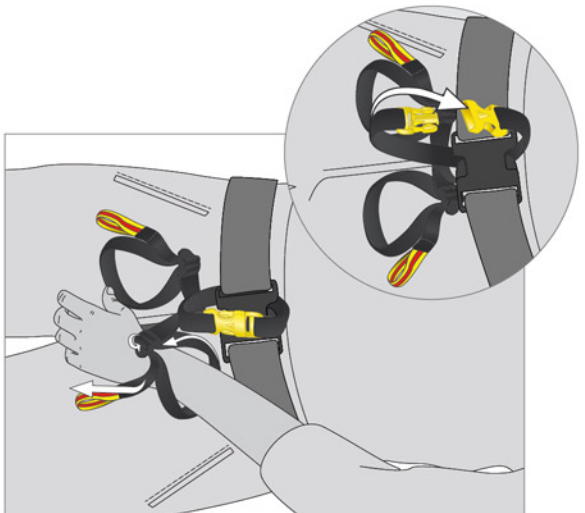
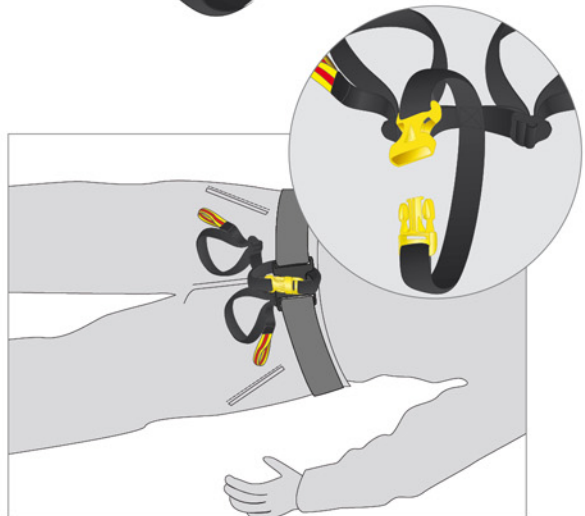
When the loop is fixed the hand loops are positioned below the waistband. Now the hands can be fixed to the hand loops. To do that the loops can be wide opened by means of the belt loops and can be tightened up after positioning of the hand (wrists) in order to avoid the hands slipping out.

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
In that way the arms can be positioned quickly and safely one after another and the evacuation of the patient can be continued.

The belt loops can be easily loosened when the loops are not needed anymore. It is also possible to open the loops.

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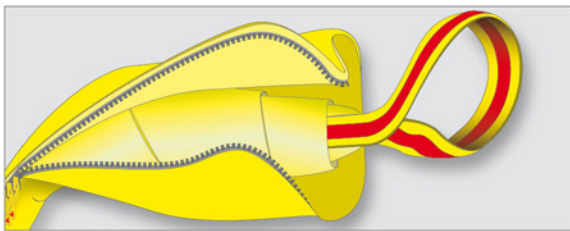The warmth blanket shows an upper and bottom side which can be distinguished from each other by the position of the loop and hook tape straps.

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In order to roll up the warmth blanket again it is unfurled flat onto the floor. The diagonal full-length loop strap marks the upside.

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Now the lifting sling with carrying slings is hooked and firmly pressed onto it full-length.

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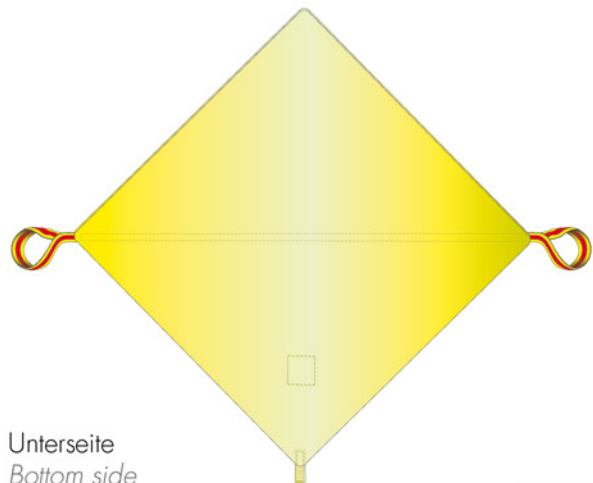
- *By removing the cover the incorporated fleece blanket can be used as emergency warmth blanket after unrolling it.*

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
- *After washing (see washing instruction) the warmth blanket can be packed into the cover again.*

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- *In the ready-to-use condition the PAX-Rescue-Boa is stored in the transport bag.*

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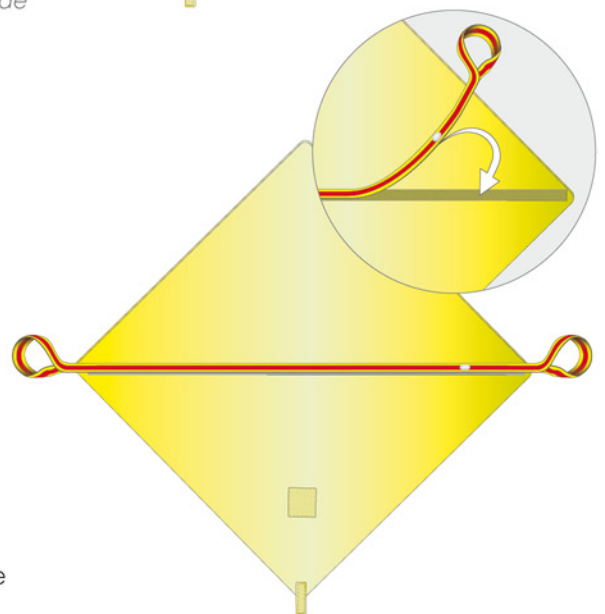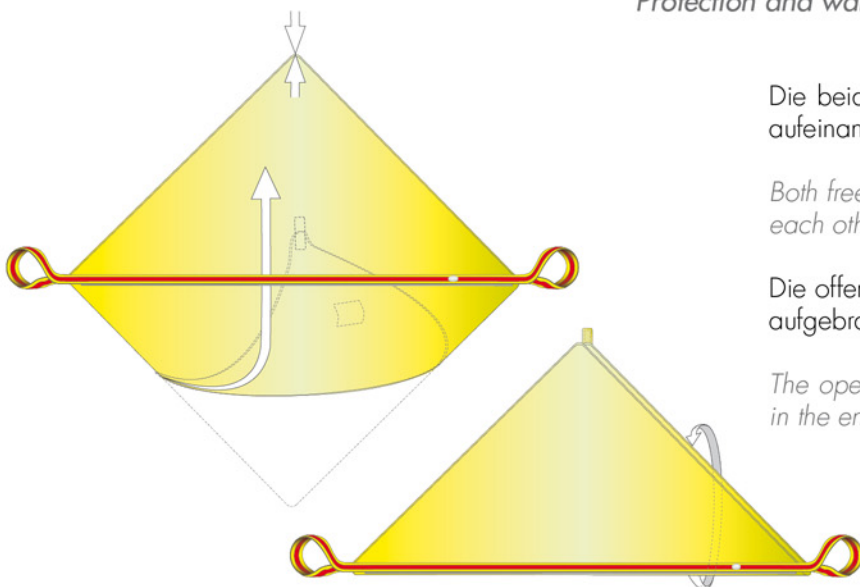
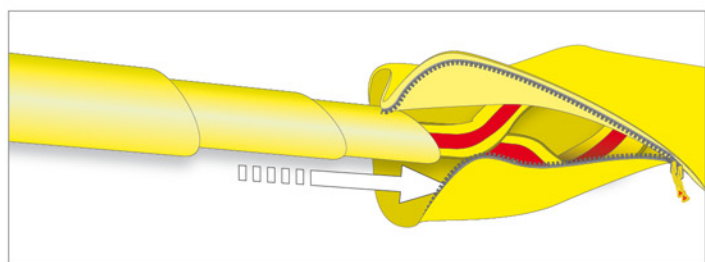
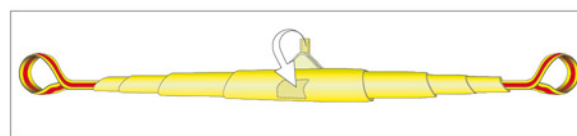
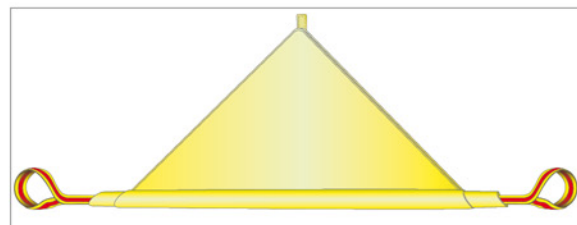
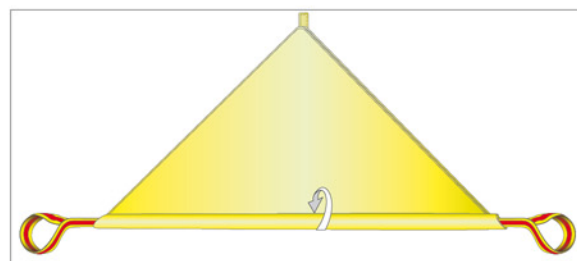
Both free tips of the fleece blanket are laid on top of each other.

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The open tip of the blanket is fixed with hook tape in the end.

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In order to put the rolled up warmth blanket with lifting sling back into the cover, the zip sliders are positioned in the centre of the cover. Now the rolled up warmth blanket is pulled through the remaining opening until the ends of the warmth blanket and the ends of the cover do match.

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After that the zip sliders are pulled to the ends. In doing so attention shall be paid to the warmth blanket not to move. The hand loops of the lifting sling protrude the same length from the ends of the cover.

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In order to put the PAX-Rescue-Boa into the transport bag both ends are lead to the centre of the Boa and then both double strands are furled again. Folded in that way the Boa fits well into its transport bag.

Für den Waschvorgang werden Wärmedecke, Überzug und Hebegurt voneinander getrennt.
As far as the process of washing is concerned the warmth blanket, cover and lifting sling are separated from each other.

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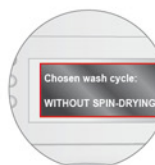
- *The max. washing temperature should be 60 degree Celsius.*

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- *The cover of the PAX-Rescue-Boa may not be spin-dried.*

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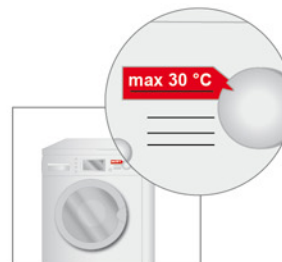
- *The max. washing temperature should be 30 degree Celsius.*

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- *The warmth blanket of the PAX-Rescue-Boa may not be spin-dried.*

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The product has a lifetime of max. 5 years from purchase date.

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Every year of use the person in charge of medical products has to carry out a visual inspection of all seams, connections, handles and surface material and to record it at the end of these instructions for use. Defective products may not be used anymore and have to be disposed of.

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Still open questions?

In case you do have open questions about the product or about another product or if you can imagine a special solution or if you want to learn more about us: We are there for you!

Phone:

*By this number you are calling PAX directly! Neither a call center nor a never ending loop.
Phone: +49 (0)44 07 - 7 14 76 - 0*

Internet:

*Where ever you are on the way: Our website with integrated shop is opened for you 24 hrs a day, 7 days a week.
www.pax-bags.de*

Email:

*You would like to send us a note, an inquiry a message? Under the below address you are right. From there your corresponding contact person will be determined.
info@pax-bags.de*

Home:

*Our head office is located in Northern Germany and if you would like to write to us, the letters have to be addressed to:
X-CEN-TEK GmbH & Co.KG
Westerburger Weg 30
26203 Wardenburg /Germany*

facebook:

*You want to be part of our community and have a say. Then call in here:
www.facebook.com/Paxbags*

Ideas:

*You do have a good idea and you miss a partner at your side to put into practice your bag project?
Tell us about it!*

Photos:

*Our bags are around a lot and you have released the shutter? Here you can send your best photos to and maybe we use them for our next catalogue.
fotowettbewerb@pax-bags.de*

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Product information:

All rights reserved. Texts, images, graphics are protected by copyright and other protective laws. The author assumes no responsibility for the topicality, correctness, completeness or quality of the information. Liability claims against the author which refer to damages of material or immaterial nature caused by use or disuse of the presented information, respectively by the use of faulty and incomplete information are principally excluded. The author explicitly reserves the right to alter, to supplement, to delete parts of the pages or to temporarily or permanently cease publication of the pages. A proliferation for commercial purpose is prohibited. All rights are with X-CEN-TEK GmbH & Co. KG. The drawings raise no claims to legal obligation or to the style of the original to the real products.

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